



Blade Remover Directions for Use

Cincinnati Surgical Blade Remover Part number: 48

This blade remover requires a two-hand process.



Step 1
Insert blade with the cutting edge of the blade facing left.

Step 2
Press down on the top the box where it says "press"

Step 3
Pull back on the handle while pressing down on the top of the box. This will disengage the blade and it will drop into the box.

Cincinnati Surgical, through our partnership with Swann Morton, brings global leading surgical blade manufacturing to the US market. From the quality of the steel to our ISO manufacturing standards and visual inspection of each blade, we are able to provide a consistent, zero defect product line that will ensure durability and consistency each time it is used. With our extensive range of products and ability to ship same day, we offer our customers a product they know they can trust when quality counts.



Raising the Standard™

Learn more at:
CincinnatiSurgical.com